User guide

<https://emagina.care/notice>

P.01 EN/ENGLISH


P.06 FR/FRANCAIS

P.11 DE/DEUTSCH

P.16 ES/ESPAÑOL

P.21 IT/ITALIANO

V1 - December 2025

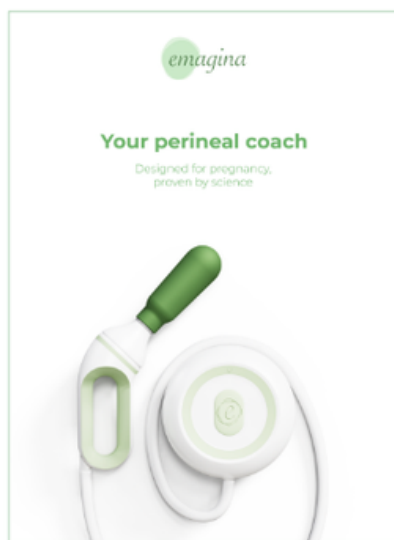Read all the instructions before using the product.

Failure to follow the instructions and safety information may result in injury to the user or damage to the device.

1. Product description

1.1 Intended use

Emagina is intended to be used by adult women who are pregnant for the first time to improve the flexibility of their perineum in preparation for their first delivery. Emagina should be used by pregnant women from the 20th week of amenorrhoea until delivery, provided there are no contraindications.

1.2 Composition

The Emagina device is composed of the following materials:

- 1) an anatomically shaped inflatable silicone balloon
- 2) a plastic handle
- 3) a flexible silicone tube
- 4) a plastic disc containing electronic components and a motor powered by 6 AA FR6 L91 LITHIUM ULTIMATE 1.5 V BL4 batteries (included)
- 5) a mobile app
- 6) a transparent silicone case
- 7) a cardboard sleeve

If the packaging is not sealed or if the cardboard box is damaged upon delivery, do not use the product and contact the seller.

The Emagina device connects to your mobile app via Bluetooth. The Bluetooth Low Energy (BLE) connection uses low-intensity, low-power waves.

A single-use condom may be used for each session. The Emagina device does not include condoms.

1.3 Useful life

The service life of an Emagina device is one (1) year from the first use, provided that all conditions of use, storage, and cleaning have been respected. If the product is damaged or compromised in any way, it is essential to dispose of the device. The shelf life of the EMAGINA device before any use is two (2) years.

2. Contraindications, warnings, and precautions for use

2.1 Contraindications

You should refrain from using the Emagina device in the following cases:

1. Non-pregnant women
2. High-risk pregnancy
3. Scheduled caesarean section
4. Pelvic abnormalities
5. History of vaginal or perineal surgery
6. Identified risk of premature birth
7. Use of Epi-No®
8. Planned home birth
9. Nerve damage and other conditions that decrease sensitivity to pain in the genital area (paraplegia, multiple sclerosis, diabetic neuropathy, taking medications that alter sensitivity in the genital area).
10. Collagenosis or other chronic disorders affecting collagen
11. Latex allergy
12. Allergy to silicone-based lubricant
13. Premature membrane rupture
14. Drug or alcohol abuse
15. If the baby is in an unusual position or has any medical condition that may make vaginal delivery impossible, e.g., placenta praevia
16. Using the Emagina device to prepare for delivery should not begin before the 20th week of amenorrhoea

If you have any questions about being able to use the Emagina device, consult your healthcare professional.

2.2 Warnings and precautions for use

2.2.1 Precautions to be taken before each use of the Emagina device

1. Check that the balloon is not damaged before each use via the mobile app. Do not use the device if it is damaged or altered in any way.
2. Wash your hands before each use and strictly follow the recommended cleaning instructions.
3. For hygiene reasons and to avoid infection, the Emagina device should only be used by one woman. Otherwise, there may be a risk of cross-contamination and infection.
4. Stop the training exercises if you feel pain and consult your healthcare professional.
5. It is recommended that you hold the handle firmly when using the Emagina balloon.
6. If using a condom, it must be placed over the device before use.
7. You should stop using the Emagina device and contact your healthcare professional if you experience vaginal bleeding.

Contact your healthcare professional if you experience any of the following symptoms:

- Fever
- Urinary tract infection
- Candidal vaginitis
- Bacterial vaginosis
- Risk of premature birth
- Low placenta
- Natural childbirth becomes impossible
- Premature membrane rupture
- Contraindication for sexual intercourse

2.2.2 Warnings about using the Emagina device

- Keep the Emagina device in good condition and do not expose it to sunlight.
- Keep the Emagina device out of reach of children.
- Do not use the Emagina balloon when in water.
- Do not expose the Emagina device to water or excessive moisture.
- Do not put the Emagina device in the microwave.
- Do not drop the Emagina device. If the device does fall, start a verification phase (see - Step 4: Start verification).
- Do not use the Emagina device after its expiry date.
- Remove the Emagina balloon before using the restroom.
- Do not clean the Emagina device with abrasive products.
- Do not put the Emagina device in the dishwasher or washing machine.
- Do not insert the Emagina balloon into any part of the body other than the vagina.
- Do not have sexual intercourse with the Emagina balloon inserted.
- Do not use the Emagina device near sharp objects.
- If the device suffers a significant impact, check that the central deflation button is working properly by pressing it during the verification phase.

Any serious incident related to the device must be reported to the manufacturer of the medical device.

2.3 Clinical performance and benefits

Through this training, the Emagina device aims to improve the flexibility of the perineal muscles and prepare them for the first delivery. Flexibility of the perineal muscles can help women giving birth for the first time prevent pelvic trauma during delivery.

2.4 Undesirable side effects

The expected side effects related to using the Emagina device are:

1. Discomfort: Any device inserted into the body may cause discomfort or pain. This may be especially true for pregnant women, who may experience changes in sensitivity.
2. Local irritation and infection: If the device is not cleaned properly, then dirt, bacteria, and viruses introduced by the device may cause local irritation and infection.
3. Injuries and bleeding of the vaginal mucosa: Incorrect use of the device may cause injuries and vaginal bleeding.
4. Psychological impact: Some women may experience anxiety or stress related to using the Emagina device.
5. Air embolism: If the balloon is perforated, a large amount of air may enter the body.
6. Premature labour: If used in contraindicated conditions.

In the event of an incident or undesirable side effects, contact your healthcare professional and the manufacturer immediately.

If the balloon is perforated: stop using the device immediately and contact the manufacturer.

In the event of an incident while using the Emagina device, contact the manufacturer immediately.

3. Before the first use

3.1 Download the Emagina app

Before the first use, download 'Emagina - Perineal Education' from the App Store for iOS smartphones or Google Play for Android smartphones. The app will guide you through each step to perform your exercises and track your progress.

The app is compatible with Android 8 and above, as well as iOS 16 and above.

3.2 Log in to the Emagina app

In the mobile app, follow the process for creating an Emagina account. Once you have completed your registration, continue with the installation to pair your Emagina device. Remember to activate pairing mode on your device. To do this:

Press the power button located on the green band of the disc; the steady blue indicator confirms that the Emagina device is on.

Press the power button on the disc again for 5 seconds. The indicator will start flashing white.

Complete the installation by accepting the pairing (iOS) or connection (Android) request that appears on your screen.

3.3 Safety information

3.3.1 Personal use

For hygiene reasons and to avoid cross-contamination and infections, the Emagina device is for personal use only.

It should be used by one woman only.

3.3.2 Quick balloon deflation button

In addition to monitoring the balloon via the mobile app, you can mechanically deflate the balloon at any time by pressing the deflation button (6) located in the centre of the disc.

The device will remain inactive until the disc is completely restarted.

If the device suffers a significant impact, check that the central deflation button is working properly by pressing it during the verification phase.

4. Before each training session

4.1 Clean the Emagina device

Before each training session, clean the Emagina handle and balloon according to the recommendations in Chapter 6 - Cleaning Procedure.

4.2 Use the notch to fit the condom over the balloon

Before each training session and after washing it, you may place a single-use condom over the Emagina balloon.

Use the notch (7) on the handle to secure the condom in place.

4.3 Start the session

4.3.1 - Step 1: Turn on your Emagina disc

Press the power button located on the green band of the disc.

A solid blue light confirms that the Emagina device is turned on.

4.3.2 - Step 2: Go to the Exercises tab

In the Emagina app, go to the 'Exercises' tab.

Explanation of the pictograms:

- 
- he 'Home' tab allows you to return to the home page, where you will find your progress and the sessions you have left to do this week.
 - Click on the 'Exercises' tab or the 'Start' button to start a new session.
 - The 'Tracking' tab allows you to view your session history and track your progress.

4.3.3 - Step 3: Connect your mobile app to the Emagina roller

Activate Bluetooth on your phone. From the app, click on 'Connect the balloon'. The balloon will be connected in a few seconds.

A solid white light indicates that the Emagina device is connected and paired with your smartphone.

If the connection is lost during an exercise, the balloon will automatically deflate. The orange indicator will flash. See chapter 12.2 - Troubleshooting.

4.3.4 - Step 4: Start verification

Once the connection is established, start verifying that your Emagina device is working properly by clicking on the 'Proceed to verification' button. This step takes approximately 1 minute. Do not insert the Emagina balloon into the vagina during this step!

Place the handle of your Emagina device on a flat surface, such as your bed or a table, and proceed with the verification. Your balloon will inflate and then deflate completely. This allows you to check the integrity of the balloon and make sure that all of the components of your Emagina device are functioning properly.

If the orange light flashes: refer to chapter 12.2 - Troubleshooting.

If your device has suffered a significant impact, check that the central deflation button is working properly by pressing it during this verification phase.

The orange light should come on and the balloon should deflate within a few seconds.

Once you have completed the verification phase and have not detected any defects, you can begin your session.

5. Training exercises and instructions for use

5.1 Get comfortable before you begin

Set aside half an hour to do the training exercises at a leisurely pace in a comfortable place.

The recommended position for using the device is semi-reclining, on a bed or sofa, with your knees bent. You may also place a towel under your buttocks.

All the tutorials in the app will help you perform the training exercises and carry out the required movements correctly.

5.2 Four exercises to do with the mobile app and the balloon

5.2.1 - Exercise 1: Relax and learn to relax your perineum

For now, leave the Emagina balloon outside your body.

During this first exercise in the mobile app, breathe in through your nose and breathe out through your mouth for 1 minute and 30 seconds. Relax all your other muscles, including your abdominal and gluteal muscles.

This exercise should only be done during pregnancy, as part of your preparation for childbirth.

To exhale correctly, pretend you are blowing out a candle with your lips pursed.

Follow the instructions in the Emagina app by clicking on 'Start'.

If you would like to place a condom over the balloon, refer to chapter 4.2: use the notch to place the condom over the balloon.

5.2.2 - Exercise 2: Warming up the perineum

Remain in the same semi-reclining position and start Exercise 2 in the app. To facilitate insertion, the balloon will inflate automatically before you begin the exercise. Then, insert the balloon halfway into the vagina, holding the handle firmly. Warm up the perineum by following the instructions in the app. Remove the balloon from the vagina at the end of the exercise.

During this exercise, it is important to learn how to relax the perineum. Breathe normally, do not block your airways and make sure to keep breathing.

To change the size of the balloon: press the '+' or '-' button at the bottom of the screen.

To contract your perineum correctly, pretend you are holding in gas. Do not tense your legs, abdomen, or gluteal muscles.

5.2.3 - Exercise 3: Perineal flexibility

There are three possible positions for this training exercise:

- Position 1: Lie on your back with your knees bent and your feet flat on the floor.
- Position 2: Lie down with your legs straight out.
- Position 3: Lie on your right or left side.

To facilitate insertion, the balloon will inflate automatically before you begin the exercise. Then, insert the balloon completely into the vagina, holding the handle firmly.

First session: inflate the balloon as much as possible until you feel discomfort. Relax and loosen your pelvic muscles before gently removing the balloon using the handle. Repeat this insertion and removal several times, up to a maximum of 5 removals.

Subsequent sessions: inflate the Emagina balloon to a diameter slightly less than the maximum diameter of the previous session. Perform 5 insertions and removals, gradually increasing the diameter of the balloon, as indicated in the Emagina app.

When removing the balloon, relax your pelvic muscles and blow out through the mouth while removing the balloon, as in Exercise 1. Do not clench your perineum or push!

5.2.4 - Exercise 4: Loosening the vaginal entrance

This training exercise will be automatically suggested to you in the mobile app 75 days after you start your training.

- Hold the handle to insert the balloon and keep half of the Emagina balloon in the vagina.
- Slowly inflate the Emagina balloon until you feel a stretch.
- Then, slightly deflate the Emagina balloon without removing it from your vagina.
- Re-inflate the Emagina balloon very slightly until you feel a stretch again.
- Repeat this training exercise as indicated in the Emagina app.

Relax during the exercise! The more you practise, the easier the exercises will become and the more positive results you will achieve.

5.3 After completing the exercises, clean and store the device

At the end of each training session, remove the condom and dispose of it.

If the device does not switch off automatically: press the button on the green band of the disc for 2 seconds.

Clean the Emagina handle and balloon according to the recommendations in Chapter 6 - Cleaning Procedure.

Store the Emagina device in its plastic bag and cardboard packaging.

Store the Emagina device in a dry place, protected from sunlight or any other heat source, at a temperature between 10°C and 40°C, out of reach of children.

6. Cleaning procedure

Do not disassemble the device for cleaning.

Clean the Emagina handle and balloon before and after each use:

- Wash your hands thoroughly with soap and water.
- Rinse and scrub the balloon and handle with hot water for 30 seconds.
- Wash with a small amount of liquid soap for 1 minute.
- Rinse and scrub with warm water for 1 minute.
- Leave the balloon and handle to dry on a clean, absorbent surface.

The Emagina disc is not designed to be protected in the event of the device being submerged in water. Ensure that you keep the Emagina disc dry during cleaning, as water could damage the device.

7. Storage, transport, and disposal

7.1 Storage and transport conditions

Store the Emagina device in a dry place, protected from sunlight or any other heat source, at a temperature between 10°C and 40°C, out of reach of children and protected from sharp or pointed objects.

Transport the Emagina device with care. Do not drop it.

7.2 Disposal conditions

The batteries and the Emagina device must not be disposed of as normal household waste. It is your responsibility to dispose of this item by taking it to an authorised collection point for recycling electrical and electronic equipment. Consult your local authorities for the location of these facilities. Please note that the lithium batteries in the disc pose a fire hazard and may explode if incinerated.

8. Technical information

8.1 Emagina mobile app and device versions

Currently, the Emagina V1.0 model of the Emagina device is compatible with version 1.1.10 of the Emagina mobile app.

8.2 Troubleshooting

The indicator light on the Emagina roller works as follows:

- Solid blue: The Emagina device is turned on.
- Flashing white: The Emagina device is ready to pair with a smartphone.
- Solid white: The Emagina device is connected and paired with your smartphone.
- Flashing orange: A fault has been detected (e.g. low batteries, punctured balloon, component failure); refer to the message displayed on the mobile app and the table below.

App and device errors		
Error code	Problem description	Action
1XXX (1000 to 1999)	Problem creating an account or logging into an existing account	Restart the account creation or login process using the correct credentials.
2XXX (2000 to 2999) 3XXX (3000 to 3999) 4XXX (4000 to 4999)	Various issues when using the mobile app that do not affect user safety or the status of the Emagina device.	Close the error message, exit the app if necessary, and resume normal use.
5XXX (5000 to 5999)	Problem with Bluetooth communication between the mobile app and the Emagina device.	Restart the Emagina device and resume normal use.
6XXX (6000 to 6999)	Defective Emagina device component	Restart the Emagina device and run a verification phase. If the error persists, contact your distributor or the manufacturer.

In case of battery failure: do not replace the batteries; if your Emagina no longer turns on, contact the manufacturer.
Contact the manufacturer for assistance if you encounter any difficulties with the operation of the Emagina device

8.3 Emagina device features

The Emagina device is designed for home use.
It is a medical device powered internally by 6 1.5V AA FR6 L91 batteries.
Modifying the Emagina device is prohibited.
The essential function of the Emagina device is to ensure the safe and controlled inflation and deflation of the balloon. In the event of failure, the mechanical valve allows for manual deflation to ensure patient safety. Its condition and correct operation must be checked whenever the device suffers a significant impact.
Essential functions include:

- The absence of activation of the EMAGINA device without request.
- No degradation of the EMAGINA device when exposed to electromagnetic disturbances.

The balloon is the applied part of the EMAGINA device, type BF, as it comes into direct contact with the vaginal mucosa to ensure the intended clinical function.

8.4 Electromagnetic compatibility and warnings

The radio frequency emissions of the Emagina device are very low and are unlikely to cause interference with nearby electronic devices.
The Emagina device is suitable for use in home environments.
The Emagina device uses wireless communications in the following frequency bands: Bluetooth BLE: [2.400 GHz - 2.485 GHz],
Avoid using the Emagina device near other equipment or stacked with other equipment, as this may lead to malfunction. If it is necessary to use it in this way, monitor this equipment and other equipment to ensure that they are functioning normally.
The use of accessories, probes, or cables other than those supplied by the manufacturer may increase electromagnetic emissions or decrease the electromagnetic immunity of this equipment, thereby causing it to malfunction.
Portable radio frequency communication equipment should be used at a minimum distance of 30 cm (12 inches) from any part of the power supply or battery charger, including cables. Otherwise, the performance of this equipment may be affected.
EMAGINA is suitable for use in the following environment. The user must ensure that it is used only in the specified electromagnetic environment. The EMAGINA device complies with the requirements of the following standards:

Emission test	Compliance	Electromagnetic environment guidance
RF emissions - CISPR 11	Group 1	EMAGINA uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment
RF emissions - CISPR 11	Class B	EMAGINA is suitable for use in all establishments including domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic current emissions - IEC 61000-3-2	Not applicable	/
Voltage changes, voltages fluctuations and flicker EMISSIONS- IEC 61000-3-3	Not applicable	/

Immunity tests	IEC 60601-1-2 ed.4.1 Compliance	Compliance level	Electromagnetic environment guidance
Electrostatic discharge (ESD), IEC 61000-4-2	±8 kV contact ±15 kV air	±8 kV contact ±15 kV air	Floors shall be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity shall be at least 30 %
Radiated RF electromagnetic field, IEC 61000-4-3	10 V/m 80 MHz to 2.7 GHz 80% AM at 1kHz	10 V/m 80 MHz to 2.7 GHz 80% AM at 1kHz	Potable and mobile RF communications equipment should be used no closer to any part of EMAGINA, including cables.
IMMUNITY to proximity fields from RF-wireless communications equipment, IEC 61000-4-3	Levels of table below	Levels of table below	
Conducted RF, IEC 61000-4-6	Not applicable	Not applicable	
Electrostatic fast transient / burst, IEC 61000-4-4	Not applicable	Not applicable	-
Surge, IEC 61000-4-5	Not applicable	Not applicable	-
Power frequency magnetic field IEC 61000-4-8	30 A/m (50/60 Hz)	30 A/m (50/60 Hz)	Power frequency magnetic fields should be at levels characteristic of a location in a typical domestic, commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply IEC 61000-4-11	Not applicable	Not applicable	-
Immunity to proximity magnetic fields IEC 61000-4-39	9 A/m (30 kHz, CW) 65 A/m (134.2 kHz, PM 2.1 kHz) 7.5 A/m (13,56 MHz, PM, 50 kHz)	8 A/m (30 kHz, CW) 65 A/m (134.2 kHz, PM 2.1 kHz) 7.5 A/m (13,56 MHz, PM, 50 kHz)	Power frequency magnetic fields should be at levels characteristic of a location in a typical domestic, commercial or hospital environment.

Test Frequency (MHz)	Modulation	Immunity test level (V/m)
385	Group 1	27
450	Class B	28
710	Pulse modulation 217 Hz	9
745		
780		
810	Pulse modulation 18 Hz	28
870		
930		
1720	Pulse modulation 217 Hz	28
1845		
1970		
2450	Pulse modulation 217 Hz	28
5240	Pulse modulation 217 Hz	9
5500		
5785		

8.5 Material monitoring and user obligations

Users undertake to comply with the instructions, recommendations, and warnings contained in this manual.
In the event of a serious incident or the likelihood of a serious incident occurring during use of the device, users are requested to report this as soon as possible to the manufacturer and the competent authority in the country where the incident occurred.

8.6 Protection index

The Emagina medical device has a protection index of IP22, in accordance with IEC 60529.

9.Warranty

The legal warranty for the Emagina medical device covers any manufacturing and operational defects or faults for two years from the date of purchase stated on the invoice, subject to the following two conditions:

- Communication of the device's serial number.
- Use in accordance with the instruction manual.

The warranty does not cover:

- Damage due to abnormal use or use outside the scope of the device's intended purpose.
- Damage caused by negligence or accident.
- Damage caused by improper cleaning that does not comply with the instructions.
- Damage caused by a thunderstorm.
- Unauthorised maintenance or any attempt to open or repair the device.

To activate the warranty, please contact your distributor's customer service department, which will register your claim.

10. Manufacturer

Emagina V1.0 is a patented Class I medical device with CE marking in accordance with EU Regulation 2017/745. It was designed, developed, and manufactured in France by the company Mummying.

An electronic version of this instruction manual (eIFU) is available in the Emagina app and on the manufacturer's website: www.emagina.care

11. Performance characteristics

Performance characteristics		
Device dimensions	Disc	12 x 5 cm
	Tube	90 cm
	Handle and balloon	25 x 10 cm
Total dimensions		25 x 15 x 10 cm
Balloon dimensions	Deflated	100 x 40 mm
	Inflated to max diameter	140 x 100 mm
Balloon pressure resistance		4
Balloon inflation resistance		Diameter ≤120mm
Power supply voltage		9 V
Type of communication with smartphone		Bluetooth Low Energy
Smartphone compatibility		iOS 16+ / Android 9+
Mobile app name		Emagina
Conditions of use and storage	Humidity	50% to 90%
	Pressure	0.7 to 1 bar
	Temperature	10° to 40°C

12. Explanation of label images

Image	Meaning
	Manufacturer
	Date of manufacture
	Expiry date
	Serial number
	Lot number
	Type BF electrical shock protection rating Applied part (type BF)
	See instructions for use
	Warning
	Waste must be collected using a specific system and must not be disposed of in conventional waste bins
	Temperature limit
	Direct current
	Medical device according to EU Regulation 2017/745
	Single patient, multiple uses
	Store in a dry place
	Keep away from sunlight
IP22	IP22 protection index according to IEC 60529
	Unique device identifier
	CE marking indicating compliance with the Medical Device Regulation (2017/745) Class I medical device

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MUMMING SAS