# Pessary fitting set

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\*in accordance with DIN EN ISO 17664  
Sterilisation of medical devices

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### Pessary fitting set

#### Product function

The pessary fitting set is designed for the healthcare professional to determine the proper size and type of pessary for the patient before prescription.

#### Caution

The pessary fitting set is not designed for patient use.

#### Contraindications

The use of a pessary fitting set is contraindicated in patients with acute genital tract infections, pelvic infections or noncompliant patients.

#### Procedures

Fitting usually requires a trial of various sizes to determine the proper pessary size. The pessary fitting set is a valuable aid in selecting the correct pessary. A vaginal lubricant may be required for the patients comfort before the insertion of the pessary.

1. Perform a normal pelvic examination prior to the fitting and introduction of a pessary. A pelvic exam helps determine the appropriate size. The pessary will easily fold across the largest drainage holes to simplify insertion and removal. The pessary should be large enough for its indication, yet not cause any undue pressure or discomfort.

2. Compress the pessary and gently insert the pessary through the introitus. Once the pessary has passed through the introitus, release the compressed pessary so that it expands to its normal shape. This will insure the patients comfort and reduce the risk of pressure necrosis.

3. Gently move the pessary to the cervix to support the cystocele or rectocele and the uterus. When in the correct position, the healthcare professional should be able to insert an examining finger between the outer edge of the pessary and the vaginal wall. This spacing will insure the patients comfort and reduce the risk of pressure necrosis.

4. Once in place, the pessary should not dislodge when standing, sitting, squatting or bearing down. It should not be uncomfortable for the patient during any of these routine activities.

5. When fitting a pessary, be sure to instruct the patient on its cleaning, insertion and removal. The patient should perform these tasks before the first follow-up visit.

6. To remove the pessary, carefully move the pessary forward towards the introitus. When the pessary is within reach, compress the pessary and withdraw gently with caution.

The healthcare professional should determine if the patient requires oestrogen therapy before prescribing a pessary.

#### See first page for disinfection/sterilisation

#### Check

Each pessary should be periodically checked for deep cracks or scratches that could make adequate cleaning and disinfection or sterilization impossible.

A pessary with any deep cracks or scratches should be discarded and replaced immediately.

The pessary fitting set is validated for at least 150 cleaning and sterilizing cycles.

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